




Refill CSM monitor from CSM code

BS% -?- -?-
EMG% -?- -?-
SQI% -?- -?-
BAT 0:19:34



1

BS% -?- -?- CSI
Menu Alarm Config
Device information




2

BS% -?- -?- CSI
Device information
Link Log
Code

3


BS% -?- -?- CSI
Code
Enter code
8888.8888.8888.8888
ID: 1

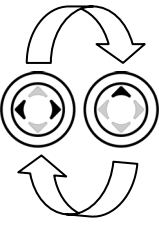

16 x


4

BS% -?- -?- CSI
Code
Code accepted
87AC.19D8.F246.ED12
100 sessions added



5


Refill CSM monitor from CSM Dispenser

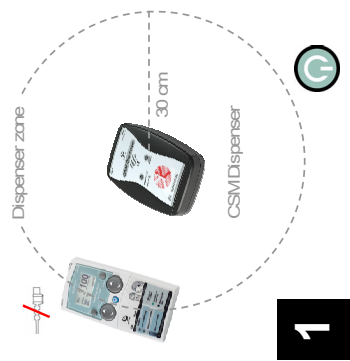

Dispenser zone
30 cm
CSM Dispenser


1

Dispenser 200
Press Set
Before refill 11
After refill **25**



2

Dispenser 186
Press Set
Before refill 25
After refill **25**



3

CSM Session Refill Guide

Options and values shown subject to CSM Dispenser setting.
Actual values all by example
Consult the CSM User Manual
Consult the CSM Dispenser User Manual




F E D C B A 9 8 7 6 5 4 3 2 1 0

87AC.19D8.F246.ED12

562700-RG


Refill CSM Dispenser from CSM code

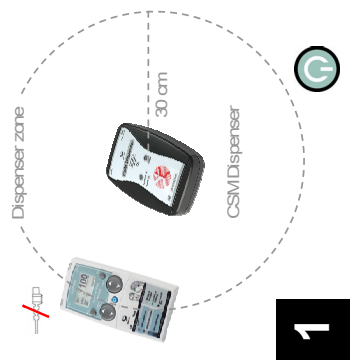

Dispenser zone
30 cm
CSM Dispenser



1

Dispenser 0
Press Set
Before refill 11
After refill **25**




2

Dispenser 0
Settings Max. number
Code

3


Dispenser 0
Settings Enter code
8888.8888.8888.8888
ID: 1

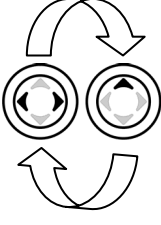

16 x

4

Dispenser 400
Settings Code accepted
87AC.19D8.F246.ED12
400 sessions added



5

! ID number MUST match ID number on code letter